

Intimate Gel

LeiLuiGel is a medical device used in the woman and man for the treatment of inflammations found in the area of the vaginal and anal mucous membranes.

It is capable to create a barrier-forming, restructuring, natural and breathable that protects the mucous membranes from external agents. Natural substances that are present in its formulation combine to restore the physiological mechanisms involved in the regeneration of the mucous membrane processes.

The function guaranteed by the Allantoin barrier is supported by the action of the acid 18 β -Glycyrrhetic soothing, antibacterial, antifungal and antiviral properties of tea tree oil (Melaleuca oil). The Chlorhexidine performs as a disinfectant helping to protect the area from infection.

Indications

LeiLuiGel is a soothing product that can be used in cases of:

- Vaginal dryness;
- Localized plagues in the sphincter anal area;
- Anal fissures;
- Lesions of the external anal sphincter;
- Irritation of the vagina and the penis.

In particular, in the vaginal irritations, LeiLuiGel promotes the regeneration of the normal conditions of altered area and the elimination of the causative agents.

Directions for use

Apply the gel evenly 2/3 times a day over the infected/irritated area. Before applying the gel, make sure that the mucous membranes are well washed and dry. Allow to air dry.

Warning: do not exceed the prescribed dose without your doctor's advice. If irritation occurs discontinue use of the product.

Contraindications and side effects

There are no known overdose phenomena. It should not be used in case of known hypersensitivity of one or more components of the formulation. For any adverse effect contact your doctor or pharmacist.

Precautions

Avoid contact with eyes, in case of accidental contact, rinse with plenty of water. Do not swallow. Keep out of children.

Do not use the product during pregnancy, lactation and in children without consulting your doctor's advice.

Shelf life and storage

Keep in a cool, dry place away from heat sources.

Do not use after the expiration date on the package.

The expiry date refers to the sealed container when stored.

Composition

Aqua, Hydroxyethylcellulose, Melaleuca Alternifolia (Tea Tree) Leaf Oil, Allantoin, Chlorhexidine Digluconate, Glycyrrhetic Acid.

Validity

Use before: see date of expiry

Packs

50 ml Airless Dispenser bottle

Category

Medical Device Class I

Product owner and distributor

L.N.Age S.r.l.
Piazza Bologna, 1
00168 Roma - Italia
www.lnage.it - info@lnage.it

Manufactured at

Via Indro Montanelli, 131-133
00168 Roma - Italia



Made in Italy

Last revision: January 2017

EXTERNAL USE